## De-Bug Wheelchair First Use and Precautions for Use

Date
------

## First Use:

- 1. As is the case with classic wheelchairs, there is a period of adaptation for using the De-Bug.
- 2. The exterior all terrain wheels serve as a hand rim for self-propelling users.
- 3. The balloon wheels used for the beach require the assistance of a carer/personal assistant.
- 4. The De-Bug wheelchair is designed for a maximum weight of 158kg = 25 stone.

## **Precautions for Use:**

- 1. The De-Bug wheelchair **is not a boat**. It allows entry into the water **up to the height of the wheelchair axle** and is not designed to float with the user in it.
- 2. Do not immerse the De-Bug wheelchair water must not go above the height of the rear axle.
- 3. Wearing a buoyancy aid is obligatory if entering the water. It is also obligatory to remain within the designated swimming area noted by signs located on the beach.
- 4. A carer/personal assistant is required for entering the water.
- 5. Users must be pushed/assisted exclusively by capable adults (especially when balloon wheels are in use).
- 6. The utmost caution is recommended on beaches and steep slipways (apply breaks).
- 7. Do not use the De-Bug wheelchair when a wheel is punctured or flat.
- 8. Using the De-Bug wheelchair with more than one person in the seat **is prohibited**.
- 9. Standing up on the seat or the footrest guard **is prohibited**.
- 10. Modifying the structure of the wheelchair **is prohibited**.
- 11. Avoid going under obstacles with the De-Bug wheelchair.
- 12. Avoid hand-braking on steep surfaces due to the risk of heating and injury.
- 13. Do not move about on inclinations greater than 15%.
- 14. Do not attempt to clear obstacles larger than 10cm.
- 15. Do not smoke while using the wheelchair, the seat is designed not to catch fire (within 15 seconds) on contact with cigarettes or matches, allowing the user time to react.
- 16. You must always be accompanied for entering water, or when the requirement of a harness is in use.

I have read/listened, understand and agree to abide by the terms of use of the De-Bug wheelchair.

## **Mandatory**

Signed	Printed	Mobile No.
	- IIIICG	MODIIC 140.